## **Data Sheet**



## **THERMARMOUR Hypothermia Bag**

<b>Product Code</b>	OB600236150OVEM52	
GTIN/UDI	05060885279361	
Colour(s)	Olive Green Outer with Grey Base	
Size	230cm x 56cm	
Material	THERMARMOUR Xtreme 8-layer laminated fabric	
Care	Launderable up to 40 degrees	
Buy Online	www.interweavetextiles.com/products/patient-warming/	

Designed with the goal of maintaining and restoring normothermic temperatures in emergencies, THERMARMOUR is suitable for patients at risk of hypothermia. Other scenarios include, but are not limited to, pre-warming for surgeries and temperature management of trauma casualties.

Key Features	*	Latex Free, Recyclable & X-ray Compatible.
	*	Easily transportable standalone device.
	*	No waiting to use once opened; a bonus for first responders.
	*	A cost and time efficient passive warming method.





# **Storing and Handling**

Keep dry and store out of direct sunlight until needed. THERMARMOUR products are for single patient use and should be discarded once soiled. If there is any damage identified upon receipt, please contact sales@thermarmour.com before use.

#### **Risks and Precautions**

Following the trial and testing period, no known residual risks or side effects have been identified. Please ensure all components including packaging are recycled or disposed of safely, as misuse may cause suffocation.

Address: Interweave House, Old Power Way, Lowfields Business Park, Elland, HX5 9DE Tel: 01422 411463 | sales@thermarmour.com | www.thermarmour.com




## **Data Sheet**



### **Usage**

Lay the patient on the inner bag base, adjusting the straps and Velcro closures to secure them. For monitoring, treatments etc, there are 4 easy access panels to mitigate fully opening it. Any patient records/key information can be shown using the transparent triage pocket. To aid with patient transportation, there are 8 carry handles as well as a supplied harness.

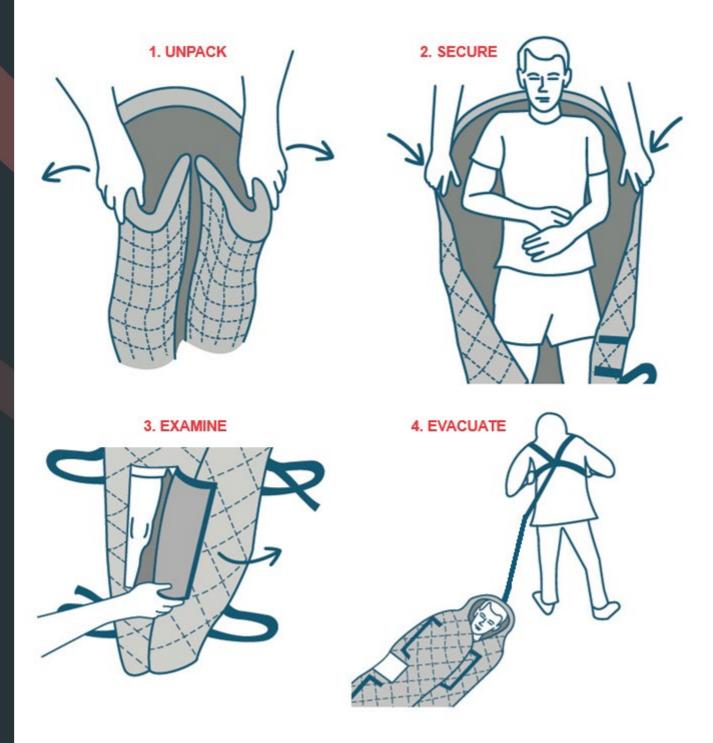

If there are any safety concerns with this device, please inform THERMARMOUR and MHRA.

Address: Interweave House, Old Power Way, Lowfields Business Park, Elland, HX5 9DE Tel: 01422 411463 | sales@thermarmour.com | www.thermarmour.com

