



Thermarmour Hypothermia Bag

- Material:** Thermarmour Xtreme material comprising 8-layer laminated fabric
- Colour:** Olive green outer / Grey base / Blue inner
- Size:** 220cm x 84cm
- Product code:** OB600236150---OVEM52
- Washable:** Can be washed at maximum 40 degrees
- Purchase online:** www.interweavetextiles.com/products/patient-warming



Thermarmour is a trading brand of Interweave Textiles Ltd.
Old Power Way, Lowfields Business Park, Elland, West Yorkshire, UK, HX5 9DE, Tel: 01422 372333

Product features

- Waterproof
- Wind proof
- Fire resistant
- Velcro limb inspection flaps
- Built-in full length absorbent mat
- Padded base layer
- Triage card pocket on outside
- Thermal cap to maintain heat in head
- Lightweight and compact
- Easy to store
- Eight lifting straps
- One carabiner clip & pulling harness
- Weight of hypothermia bag 3.6kg
- Safe lifting weight 460kg


Ideal for

- Military
- Search & rescue teams
- First responders



Thermarmour is a trading brand of Interweave Textiles Ltd.
Old Power Way, Lowfields Business Park, Elland, West Yorkshire, UK, HX5 9DE Tel: 01422 372333


Instructions for use

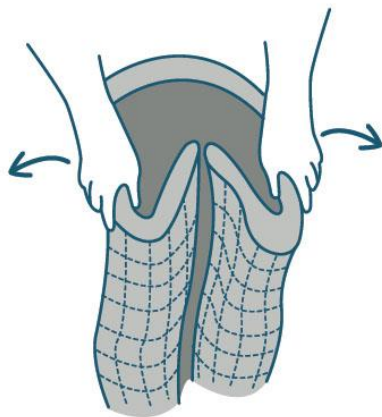


1. Unpack the hypothermia bag and spread it out with the grey side down.




2. Lay the patient on the inner base. Lift the sides of the hypothermia bag, close the bag and adjust the closure with the velcro. The patient's face can be exposed, by rolling the fabric around the face. However, full closure will prevent heat loss.

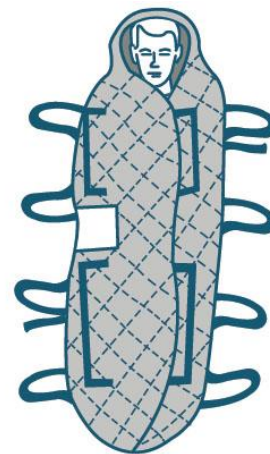

3. There are 4 carrying handles on both sides of the hypothermia bag for transporting the patient. There are two adjustable straps to secure the patient & the bag is large enough for a patient on a spine board

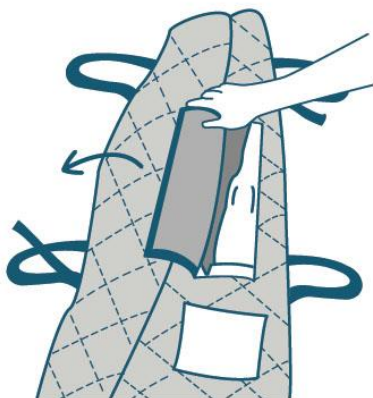

4. For examining, monitoring or treating the patient there are 4 easy access panels to view the patient without fully opening the bag.

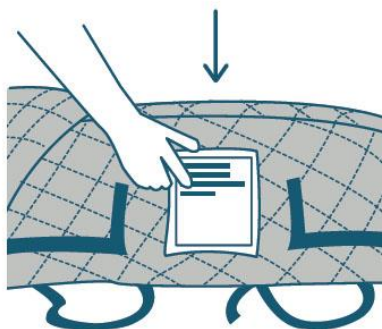

5. Use the transparent plastic pocket for casualty classification or patient records.



6. At the head end of the hypothermia bag there is a fastening loop. The supplied harness with carabiner can be attached to enable the casualty to be dragged to a safe zone.

Thermarmour is a trading brand of Interweave Textiles Ltd.
Old Power Way, Lowfields Business Park, Elland, West Yorkshire, UK, HX5 9DE Tel: 01422 372333