

## Data Sheet

### THERMARMOUR Pouch

<b>Product Code</b>	OB610-25-33---TAMM55
<b>GTIN/UDI</b>	05060885279705
<b>Colour(s)</b>	Medical Blue Outer with White Inner
<b>Size</b>	100cm x 150cm (can be cut to size if required)
<b>Material</b>	Non-conductive bilaminate foil & needle punched polyester fleece
<b>Care</b>	Patient Specific, cannot be laundered
<b>Buy Online</b>	<a href="http://www.interweavetextiles.com/products/patient-warming/">www.interweavetextiles.com/products/patient-warming/</a>


Designed with the goal of maintaining and restoring normothermic temperatures in emergencies, THERMARMOUR is suitable for patients at risk of hypothermia. Other scenarios include, but are not limited to, pre-warming for surgeries and temperature management of trauma casualties.

#### Key Features

- ❖ Latex Free, Recyclable & X-ray Compatible.
- ❖ Easily transportable standalone device.
- ❖ No waiting to use once opened; a bonus for first responders.
- ❖ A cost and time efficient passive warming method.

#### Usage

Remove THERMARMOUR Pouch from packaging.  
Place over the distressed party's extremities with the foil side outwards.  
Suitable for use on both feet and hands.

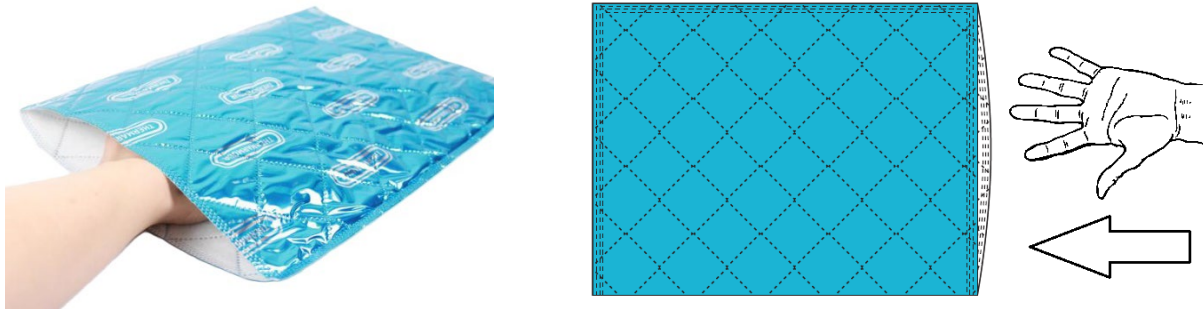

#### Packing

Manufactured in our UK facility, THERMAMOUR is a readily available, sustainable solution.


<b>Per Carton</b>	30, Bagged in 10s
<b>Carton Size</b>	39 x 29 x 21.5cm
<b>Pallet Quantity</b>	2160

## Data Sheet

### Storing and Handling

Keep dry and store out of direct sunlight until needed. THERMARMOUR products are for single patient use and should be discarded once soiled. If there is any damage identified upon receipt, please contact [sales@thermarmour.com](mailto:sales@thermarmour.com) before use.

### Technical Drawing

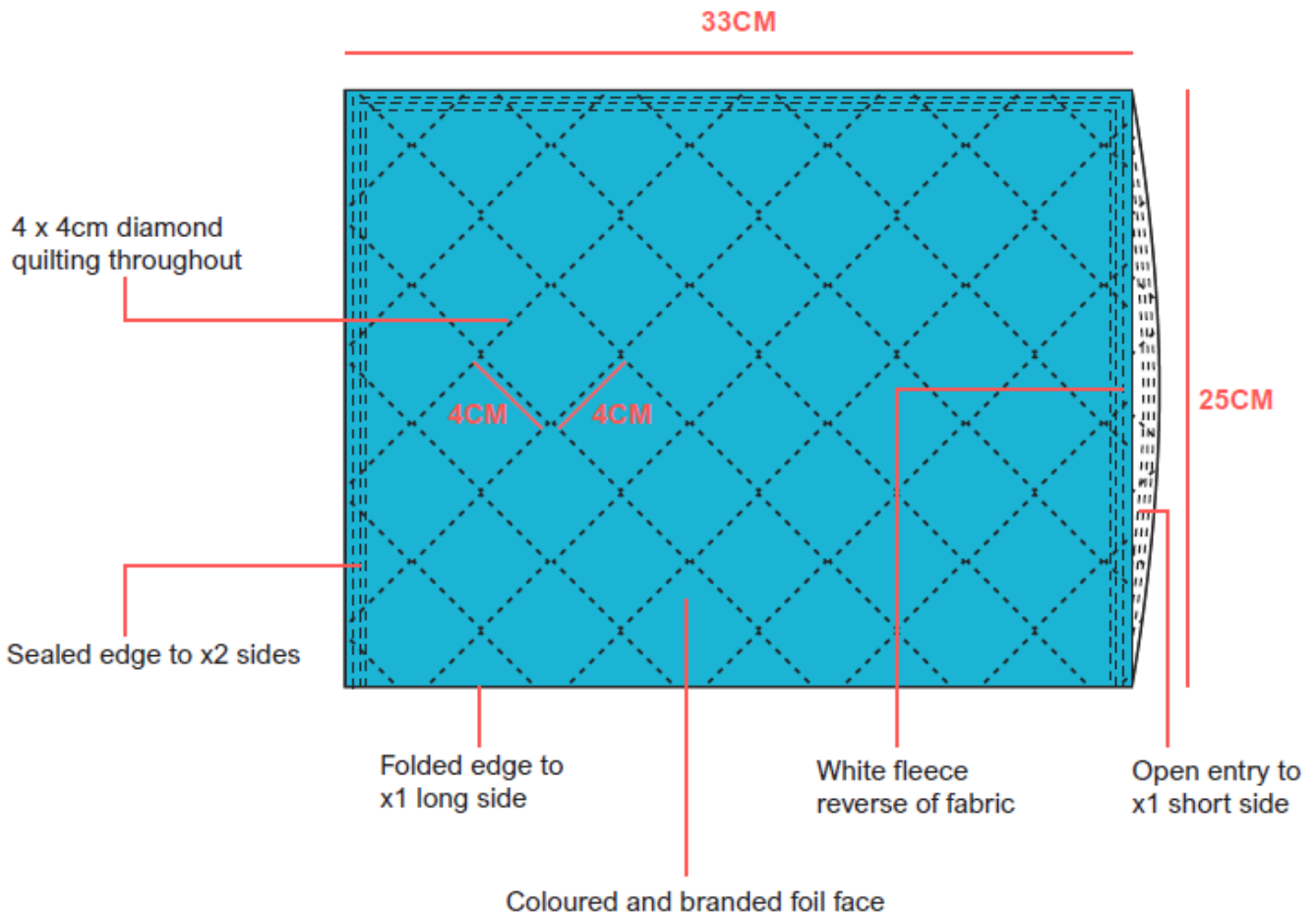

### Risks and Precautions

Following the trial and testing period, no known residual risks or side effects have been identified. Please ensure all components including packaging are recycled or disposed of safely, as misuse may cause suffocation.

If there are any safety concerns with this device, please inform THERMARMOUR and MHRA.