

THERMARMOUR Baby Wrap

Instructions For Use




Product Code	GTIN/UDI
OB202-55-57---TAEM55	05060885279682

Designed with the goal of maintaining and restoring normothermic temperatures in emergencies, THERMARMOUR is suitable for patients at risk of hypothermia. Other scenarios include, but are not limited to, pre-warming for surgeries and temperature management of trauma casualties.

Usage

Open packaging and remove the THERMARMOUR Baby Wrap. Unfold the wrap fully, placing the infant in the middle as shown. Wrap and secure with the foil side outwards.

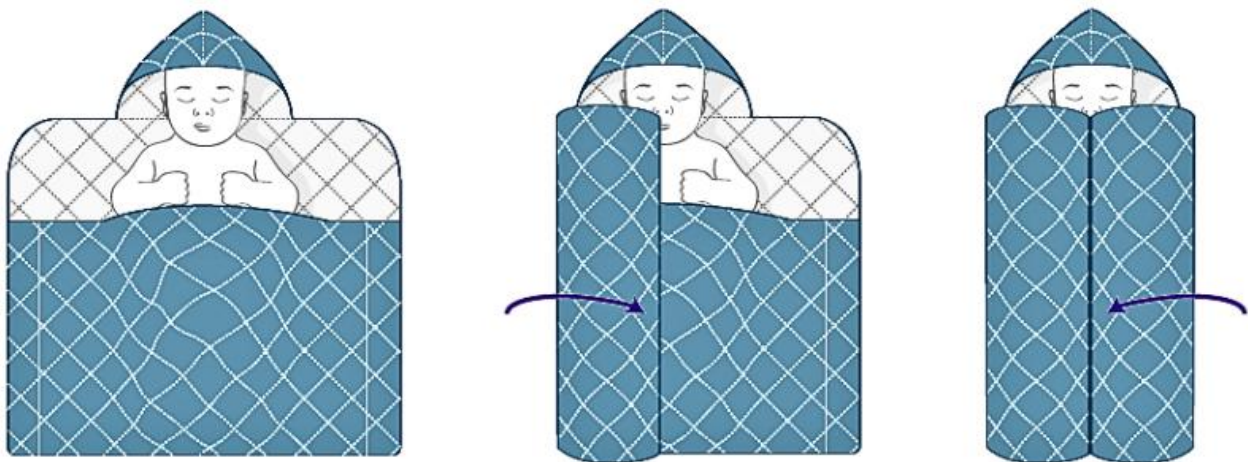

Storing and Handling

Keep dry and store out of direct sunlight until needed. THERMARMOUR products are for single patient use and should be discarded once soiled. If there is any damage identified upon receipt, please contact sales@thermarmour.com before use.

Risks and Precautions

Following the trial and testing period, no known residual risks or side effects have been identified. Please ensure all components including packaging are recycled or disposed of safely, as misuse may cause suffocation.

If there are any device safety concerns, please inform THERMARMOUR & MHRA.