

THERMARMOUR Cap Instructions For Use



Product Code	GTIN/UDI
OB620LGE-----TAMM55	05060885279378

Designed with the goal of maintaining and restoring normothermic temperatures in emergencies, THERMARMOUR is suitable for patients at risk of hypothermia. Other scenarios include, but are not limited to, pre-warming for surgeries and temperature management of trauma casualties.

Usage

Remove THERMARMOUR Cap from packaging.
Place on the patient's head with the foil side outwards.



Storing and Handling

Keep dry and store out of direct sunlight until needed. THERMARMOUR products are for single patient use and should be discarded once soiled. If there is any damage identified upon receipt, please contact sales@thermarmour.com before use.

Risks and Precautions

Following the trial and testing period, no known residual risks or side effects have been identified. Please ensure all components including packaging are recycled or disposed of safely, as misuse may cause suffocation.

If there are any safety concerns with this device, please inform THERMARMOUR and MHRA.